

MLR119 Backrest

Note


This product is to be installed by suitably qualified persons, in a fit-for-purpose application, to suitable materials, using suitable fixings, and complying with any relevant codes.

Hardware Provided

- 2x Dress Rings
- 4x Screws
- 4x Washers

Before Installation

- Inspect the product. If there is any damage or missing parts, do not install the product and contact your place of purchase.
- Ensure the wall is either solid or has adequate in-wall support.
- Ensure the wall is plumb/flat without any raised surfaces.

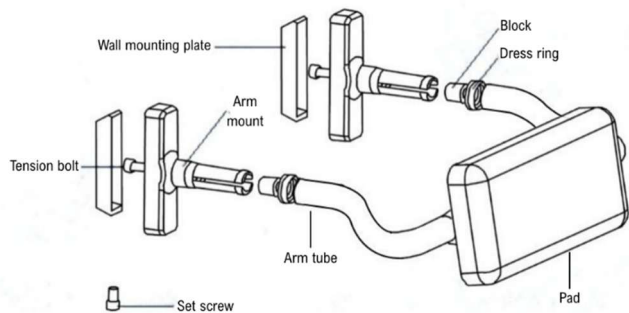

*Images depict model MLR119CMKII




This product is AS 1428.1 compliant (accessible compliant) when installed in accordance with the guidelines.

Installation Instructions

Placement may be subject to regulations/codes and architectural drawings. Correct positioning is the responsibility of the installer.

1. Undo the set screws from each arm mount. Set the screws aside. Remove the wall mounting plates from the arm mounts.
2. Place the wall mounting plates against the wall at the desired height, and checking that they are level, mark the locations of the four mounting holes.
3. Install four suitable fasteners into the wall at the marked locations and attach the mounting plates.
4. If required, the arms of the backrest can be cut down to size. After doing so, debur the tubes.
5. Remove the stickers and place a dress ring over each end of the tubes.
6. Insert the arm mounts into the arm tubes.
7. Tighten the tension bolts at the rear of the arm mounts.
8. Place the assembled rail onto the wall mounting plates.
9. Replace and tighten the set screws in each arm mount.
10. Using the screws and washers provided, and checking that the backrest pad is level, attach the pad to the rail.
11. Test backrest for correct operation.

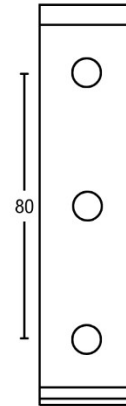

Note Regarding Accessible Compliance:

"A backrest shall ... be located with the lower edge of the centre of the face 120 to 150 mm above the WC seat and 430 to 495 mm from the front of the WC pan"

- AS 1428.1:2021 clause 12.2.4

Technical recommendations and installation advice from Metlam Australia Pty Ltd are general in nature and provided in good faith without warranty. Metlam disclaims all liability for errors, omissions, or damage resulting from improper installation or maintenance. All products must be installed by qualified tradespeople and maintained by competent personnel. It is the installer's responsibility to conduct thorough pre-installation assessments; Metlam Australia will not enter into post-installation disputes regarding positioning, functionality, or external factors affecting performance.

The photographs and line drawings of the products presented above are representational only.
Metlam Australia Pty Ltd reserves the right to, and from time to time, make changes and improvements in design and dimensions.